
Gebrauchsanweisung
Modular-Set
Manual for Modular set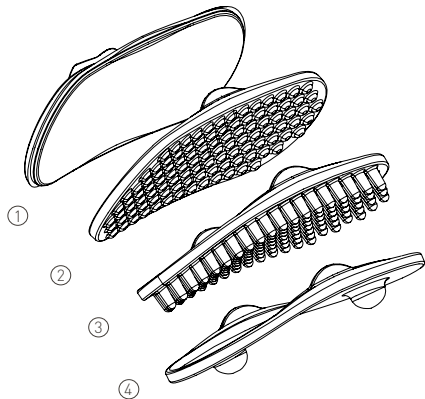

DEUTSCH

GEBRAUCHSANWEISUNG MODULAR-SET

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ENGLISH

MANUAL FOR MODULAR SET

- | | |
|----------------------------|------------------------------|
| 1. FOREWORD | 6. IMPORTANT NOTES |
| 2. SCOPE OF DELIVERY | 7. CLEANING AND DISINFECTION |
| 3. PURPOSE AND APPLICATION | 8. CONTRAINDICATIONS |
| 4. MATERIAL INFORMATION | 9. CE CONFORMITY |
| 5. SHELF LIFE | 10. CONTACT |

1. FOREWORD

The NOVAFON Modular Set serves as an extension to the existing and included attachments of the NOVAFON sound wave devices for local vibration therapy. It is intended exclusively for use in conjunction with such a device. Please read the following pages carefully to ensure proper use.

2. SCOPE OF DELIVERY

The Modular Set is available in white, anthracite grey and black (Art. no. 7028/7055/7057). The set is shown on page 1. A set consists of the following items: Oval attachment [1], 3 different click modules (grip module [2], nub module [3], trigger module [4]), instruction manual, storage bag.

3. PURPOSE AND APPLICATION

NOTE: Please also observe the instructions for use of the device when using the Modular Set. The Modular Set is primarily designed for use on animals, but can also be used on humans.

3.1 INTENDED PURPOSE

NOVAFON devices and accessories are designed to relieve pain and promote muscle relaxation for symptom relief:

- Acute pain of the musculoskeletal system
- Chronic pain of the musculoskeletal system
- Neuro-rehabilitation
- Voice and swallowing therapy

3.2 APPLICATION

When using the Modular Set, two proven treatment methods are combined: The vibrations of the NOVAFON sound wave devices to relieve pain and stimulate the upper layers of the skin to promote circulation through the specific surfaces of

the different modules. The different structures enable individual adjustment of the stimulus intensity to the treatment goal or the needs of the person concerned. The ergonomic shape is particularly well suited for use on convex parts of the body and the treatment of larger body surfaces, including the lateral back muscles, chest muscles, hindquarters and forehand muscles. Treatment close to and around bony structures (hip, shoulder) is also possible.

Oval attachment	<ul style="list-style-type: none"> · Efficient smoothing out of the body structures, right into the fasciae · Slight scraping possible over the long side · Elliptical shape allows deflecting movement
Grip module	<ul style="list-style-type: none"> · More grip · Gentle sensory input, light stimulating application · Rounded ends prevent scratching
Nub module	<ul style="list-style-type: none"> · 107 ergonomically arranged tips · Stimulating application and intensive sensory input on the skin surface · Smooth gliding even through thick fur and hair structures
Trigger module	<ul style="list-style-type: none"> · Bilateral point stimulation · Tactile two-point stimulation · Gently smoothing out of muscle strands · Intercostal stimulation (intermediate ribs)

NOTE: Vibration therapy primarily relieves the symptoms, but does not cure the underlying disease. Using the Modular Set does not replace visiting the doctor.

NOTE: The use of local vibration therapy for muscle relaxation in neurological disorders or voice and swallowing therapy should only be carried out by specialist personnel (therapists, doctors, nursing staff) or after thorough instruction by such a person.

NOTE THE CONTRAINDICATIONS in Chapter 8 and the instructions in Chapter 6. In all cases, local vibration therapy is used as one element in a comprehensive therapy plan where muscle relaxation is needed for further exercises and training

elements. NOVAFON devices can be used for animals with a body weight of 3 kg or more. When treating horses: Pay attention to occupational safety.

Can be used by people aged 6 and over. With therapeutic supervision, use is possible in patients from 2 years of age. Children must be supervised. The same may apply to people with impaired cognitive or motor control (as is often seen in patients with spasticity).

DO NOT USE in the treatment of pain or swelling of unknown origin. In this case, contact your doctor or therapist for clarification and diagnosis.

Note: All attachments can be used directly on the skin, on the fur/hair as well as through textiles. No gel or similar is needed.

3.3 ATTACHMENT ON THE NOVAFON DEVICE

The oval attachment is fixed to the device via a magnetic plug connection. The attachment is fixed by simply connecting it into the holder of the device; the intended position has been reached when the brackets on the device and the attachment lie seamlessly on top of each other. The magnets help you to find the end position. Before initiating the treatment, make sure that the attachment is locked in the intended end position. The attachment can be easily detached from the device by turning it a quarter of a turn and pulling gently.

3.4. CONNECTING A CLICK MODULE TO THE OVAL ATTACHMENT

First remove the oval attachment from the device and only then attach the click module.

Place the smooth surface of the oval attachment on the back of the selected click module. First push one of the long sides of the oval attachment into the frame of the module and fix it with the thumb at the level of the side flaps. Now press the other side of the oval attachment into the opposite tab until you hear a clicking sound. If necessary, press the second tab slightly outwards to allow the oval attachment to snap into place correctly.

3.5.CHANGING THE CLICK MODULES

First detach the oval attachment from the unit and only then remove or change the module. To change or remove the Click module, pull the two side tabs of the module apart to the side. Press one side down slightly, away from the oval attachment. There is another slight clicking noise. Then remove the other tab from the oval attachment as well. Select and attach another module as required (see Section 3.4).

4. MATERIAL INFORMATION

The NOVAFON Modular Set is made of a high-quality plastic suitable for medical applications.

5. SHELF LIFE

Shelf life may vary depending on frequency of use. All individual parts must be checked for damage and wear before use. Replace damaged or worn products early on. To avoid premature ageing of the material, store the Modular Set in a dry place away from light. Note that attachments are considered wear parts and are therefore subject to a limited warranty.

6. IMPORTANT NOTES

- Only use the Modular Set in combination with NOVAFON sound wave devices.
- Always observe all the information in the instructions for use of your NOVAFON sound wave device.
- Avoid using cream or gel.
- Be careful not to apply additional pressure, otherwise the unit may be damaged.
- Be careful of your fingers when changing the modules, there is a risk of pinching.
- To avoid premature ageing of the material, store the Modular Set in a dry place away from light.
- Check the attachments regularly for wear and damage. Damaged attachments must no longer be used. Risk of injury.
- NOVAFON accessories are only intended for use with NOVAFON sound wave devices. Use with products from other manufacturers is not permitted.
- NOTE: When using these attachments, noise production may be different from the standard attachments. Therefore, only switch on the device once you have made contact with the body.

7. CLEANING AND DISINFECTION

In professional and clinical use, the NOVAFON Modular Set must be hygienically prepared before first use (see manual cleaning and disinfection). For use on several animals or people, the attachments must be hygienically prepared after each use. Please note that machine cleaning of the Modular Set is not possible. In domestic, non-clinical use, the parts can be cleaned with a mild soapy water solution as needed or when visibly soiled.

- A. Preparation: Detach the click module from the oval attachment.
- B. Pre-treatment (if required, for example in the case of heavy surface soiling):

Required accessories: suitable plastic brush, running water (at least drinking water quality), neutral detergent or alcohol-free spray wipe disinfection (e.g. NOVACLEANER, NOVAFON GmbH) or cleaning wipes (alcohol-free, suitable for sensitive surfaces, e.g. Rheosept-WD Plus Wipes, Wachendorff-Chemie).

1. Remove surface dirt and soiling with a plastic brush under running water. Alternatively, light soiling can be removed with a separate disinfectant wipe.
2. Check for cleanliness: Repeat the pre-treatment if the dirt is still visible. Rinse each part of the attachments under running water for neutralisation and then dry with a lint-free cloth.

C. Cleaning and disinfection: Make sure that all visible soiling is removed before disinfection. Please only use alcohol-free surface disinfectants (VAH or IHO listed) for sensitive surfaces (e.g. NOVACLEANER, NOVAFON GmbH; SinAlc sensitive, Himed; Rheosept-WD Plus Wipes, Wachendorff-Chemie).

1. Thoroughly treat all surfaces of the modules and the oval attachment with the disinfectant. Make sure that all surfaces are completely wetted. Observe the instructions for use provided by the manufacturer of the disinfectant you apply.

NOTE: Due to its structure, the "nub module" attachment should preferably be cleaned by means of spray and wipe disinfection (for example NOVACLEANER, NOVAFON GmbH; SinAlc sensitive, Himed).

2. After the exposure time has elapsed, wipe dry with a lint-free cloth if necessary.

8. CONTRAINDICATIONS

Since the Modular Set is an accessory and can only be used in combination with the NOVAFON sound wave devices, the contraindications and side effects described in the instructions for use of the NOVAFON sound wave devices apply.

In addition, the following instructions must be observed:

Stop treatment if you find it uncomfortable or painful. Watch out for signs of discomfort, agitation or fear, especially when using on animals.

Please note: The appearance of slight redness and warming of the skin is not a cause for concern. This is a desired effect that results from the increased blood flow.

9. CE CONFORMITY

For this device, the manufacturer declares conformity with the requirements for class I medical devices in accordance with Regulation 2017/745/EU and UK MDR 2002 under its own responsibility. Please contact us if you suspect that the device

has caused a serious incident. As a user, you have the right to report incidents of this type to the relevant national authority (e.g. Ministry of Health).

10. CONTACT

If you have any questions, please contact your dealer or the manufacturer.

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NOVAFON Elektromedizinische Geräte GmbH,